


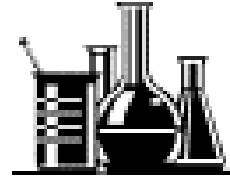

QuikTest

-Instant • Accurate • Easy to Use-



Fast, Easy Screening
Test For The
Detection Of Alcohol



**ALCO-
SCREEN™**

PROCEDURE



1. Insert test pad into mouth and saturate with saliva. Remove test pad and wait two minutes.



2. At two minutes, compare test pad with the color chart printed on the back panel of foil package. Within 15 seconds choose the closest color match to estimate the alcohol concentration.

ADVANTAGES

CONVENIENT

The ALCO-SCREEN™ can be used any time, any place. Each test is individually wrapped and completely disposable.

ECONOMICAL

ALCO-SCREEN™ greatly reduces the time and cost associated with other single and multiple use alcohol testing methods.

SENSITIVE

The relationship between the amount of saliva alcohol and blood alcohol is 1:1 while with breath it is .000048:1, making saliva a more sensitive testing medium than breath.

VERSATILE

In addition to testing saliva alcohol, the ALCO-SCREEN™ can detect the presence of alcohol in other fluids.

BENEFITS

SAFETY

ALCO-SCREEN™ promotes a safe work environment and increased productivity by helping to eliminate alcohol related accidents.

ENFORCEMENT

ALCO-SCREEN™ helps in the enforcement of “zero tolerance” laws, rules, regulations and agreements which prohibit alcohol consumption.

COMPLIANCE

ALCO-SCREEN™ helps to monitor individuals enrolled in treatment programs.